



For information in your language vetoquinol-mastatest.com/how-to-use
Deutsch, Français, English, Español, Italiano, Portuguese, Polski, Nederlands

1. INSTALL LAPBOX

- Connect to ethernet (LAN) and power using supplied cables (to connect via WiFi see page 2)
- Fold out screen by pulling from underneath (To fold away, push sliver buttons either side of the Lapbox)
- The default language is English. To change, from the main menu scroll:
 -    to *Configure Lapbox* ,   to *Select Language*  then scroll  to select 
 - Return to the main menu:     

2. REGISTER

- Login, or create an account at mastatestsystem.com portal.
- From portal menu, select *My Account*, then *Register New Lapbox*. Then input:
 - Lapbox ID (from the Lapbox main menu,   to select *Lapbox Information*)
 - Serial number (on rear of Lapbox)
 - On Lapbox, select    to return to main menu

3. CHECK CONNECTION

- Check red  and green  lights are showing on the display.
- Scroll down  to select *Test upload to Cloud*, then . If successful, Lapbox is ready to use.
-  to return to main menu.

CARE

- Operate Lapbox within room temperature range 5-30°C (41-86°F) for best performance
- Position your Lapbox on a clean, dry desk space and handle with care to protect camera inside
- Keep internal and external surfaces clean using a damp cloth (water only)
- Store Mastatest cartridges at 2-8°C (35-46°F), and use before expiry date

CONNECT VIA WiFi

(NB: ethernet is recommended - see page 1 for connecting via ethernet)

Starting from the Lapbox main menu:

1. Configure for WiFi

- Scroll down ▼▼▼ to *Configure Lapbox* **ENTER**
- Select *Set internet connection* **ENTER**
- Scroll down ▼ to *Set WiFi* **ENTER**, wait for Lapbox to restart **ENTER**

2. Set-up Network SSID

- Scroll down ▼▼▼ to *Configure Lapbox*, then **ENTER**
- Scroll down ▼ to *Setup WiFi Connection*, then **ENTER**
- **ENTER** to scan for networks, then scroll ▼ to your chosen network, and **ENTER** to select
- Go back ◀ to *Setup WiFi Connection*

NB: > -65dBm, and 2.4 GHz WiFi required

3. Enter network password

- Scroll down ▼ to *Set WiFi password*, then **ENTER**
- Input WiFi password using ◀▶ and **ENTER** to select characters.
Long-press **ENTER** when complete
- Press **ENTER** again to save
- Scroll ◀◀▶▶▶ to return to main menu

4. Check connection

- Scroll down ▼ to *Test upload to Cloud*, then **ENTER**
- ▶ to return to main menu

For information in your language:

www.vetoquinol-mastatest.com/how-to-use

Deutsch, Français, English, Español,
Italiano, Portuguese, Polski, Nederlands

